

Thank you for choosing Altitude Control Technology for your home oxygenation needs.




ALTISTAT™

BY ALTITUDE CONTROL TECHNOLOGY

A few tips to get you started using your Altistat™

- 1) **Mode selection:**
 - a. **Run** — This turns your system on. It may take 2-6 hours for your system to reach its targeted altitude. If pressing RUN from OFF, it will take approximately 7 minutes for the sensors to warm up before the system turns on.
 - b. **Standby** — This leaves your sensors on and your system will activate faster. Leave your system in this mode when using it daily.
 - c. **Off** — This turns the system completely off and powers down the sensors. Use this mode when the system won't be used for more than 5 days.
- 2) **The Altistat™ measures the effective altitude every 6 seconds and adjusts accordingly.** This typically will not match the US Geological Elevation measurement as we are looking at atmospheric pressure and oxygen content.
- 3) **Your system can be set on a schedule to automatically turn on and off before and after bedtime.** Please refer to the help center for instructions on how to set up the schedule or to set up mobile access at help.altitudecontrol.com.
- 4) **Please make sure that all doors and windows are closed to effectively use your home oxygenation system.** Some systems are designed for larger spaces that include closets and bathrooms, while some are designed for only the sleeping area. Please see your design to ensure which area was selected for your room(s).
- 5) **Extreme weather can impact the leakiness of a room.** Wind can push/pull air through homes and exchange air which will impact your room's ability to hold enriched oxygen. Extreme cold and low humidity also change the composition of many building materials allowing additional air exchange. If you notice slight changes in the effective altitude, this is normal. If your system is dramatically impacted by weather, please contact our service team for a consultation on room sealing.
- 6) **To maintain optimal performance and longevity your system should be serviced annually, even if you use your system only a few weeks out of the year.**

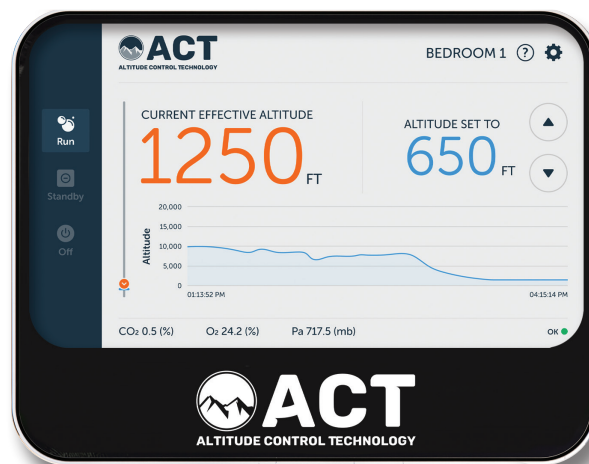

Please visit our Help Center for answers to other frequently asked questions or contact our support team Monday-Friday from 8 a.m. to 5 p.m. MST. All inquiries will be responded to within 1 business day



help.altitudecontrol.com



service@altitudecontrol.com



720-420-1077

Imagine how you'll feel tomorrow after sleeping in your oxygenated room tonight.™